
PRESTO

PRESTOS

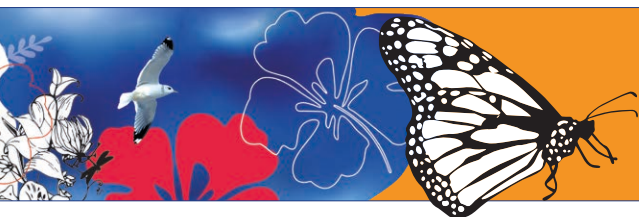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## Safety instructions

As the user of the mobility scooter, you are always responsible for complying with the locally applicable safety instructions and guidelines. It is important to first acquire driving experience under the supervision of an experienced instructor. Your dealer can advise you on this.

Life & Mobility accepts no liability for any loss or injury caused by negligence or by failure to comply with the safety instructions.

Always notify Life & Mobility and the relevant authority in case of a serious incident.

This symbol on the teller of the scooter, it means that reading the manual is necessary.

### Attention:

- If you see this symbol in the manual, it indicates a supplementary instruction. Always follow this instruction. Negligence may adversely affect the operation of the mobility scooter and may cause physical injury or damage to the mobility scooter or the environment.

### General

#### Warning:

- Never change the construction of the mobility scooter.
- Keep the mobility scooter away from open fire.
- Do not install, maintain or operate equipment without reading this manual.

#### Be careful:

- Use the basket only to transport objects.
- Do not use your mobility scooter to pull vehicles, carts, etc.
- Remember that there are protruding parts, both fixed and loose, in and on the mobility scooter. These can cause damage to the surroundings and the mobility scooter.

Information for visually impaired people can be obtained by contacting Life & Mobility, see last page for contact information.

**Attention:**

- This mobility scooter is suitable for transporting a single person.
- Only use original Life & Mobility parts and accessories.
- The mobility scooter has a fully programmable electronic control system. Changes to the programming must only be made by Life & Mobility.
- Never cover or remove the safety instructions and stickers.
- Always switch off the mobility scooter and take out the ignition key before leaving it. This prevents unwanted use and/or theft.

**Personal requirements**

**Warning:**

- If you are under the influence of substances that may affect your ability to drive, you may not drive the mobility scooter.
- You must be functional and cognitive capable of operating the scooter in its intended environment.

**Attention:**

- Your vision must be adequate to drive the mobility scooter safely in any situation that may occur during use.
- Please follow the relevant traffic rules that apply.
- Make sure that you can always be seen by other road users. As a user of a mobility scooter you are very vulnerable on the road.
- Avoid quiet routes. Make sure that help can be arranged in the event of an emergency.

**Assembling and adjustment**

(see also the chapter 'Settings and adjustments')

**Warning:**

- Before using the mobility scooter, check that all components of the mobility scooter are properly mounted and secured.

**Attention:**

- Your dealer can set up the various driving features for you with the microprocessor.

**Intended operator**

Levers only to be operated by the user are:

- throttle lever
- freewheel lever
- seat depth lever
- back angle lever

Levers to be operated by the user and assistant are:

- seat rotation lever
- hydraulic tiller lever

**Sitting in the mobility scooter**

**Warning:**

- Make sure that your fingers and limbs do not get trapped. Always be very careful:
  - When folding away armrests.
  - If you are not in full control of your arms or legs.
  - When operating the freewheel lever

**Driving the mobility scooter**

(see also the chapter 'Use')

**Warning:**

- Drive carefully on roads that are slippery due to rain, black ice or snow.
- Make sure that no loose clothing can get caught. It can obstruct the wheels and cause the scooter to overturn.

- Watch out for overturning:
  - If you have set up the seat with a large back angle.
  - When driving up a slope.
  - If the slide system is in the rear-most position.
- Do not transport anything on the handlebars.
- Do not let anyone stand on the rear of your mobility scooter.
- The scooter may come to a sudden stop at any time during operation

**Slopes and other obstacles**

**Warning:**

- Avoid slopes with a gradient greater than 10°. Driving on slopes (forwards or in reverse) is always dangerous.
- You must never free wheel down a slope. (See page 23: freewheel lever)
- Drive carefully along (pavement) edges. An uneven surface increases the risk of overturning.
- Avoid prolonged driving on inclines <math><10^\circ</math>. This can cause excessive wear on the engine and electrical components.
- When driving on a slope, the stopping distance will increase.
- The maximum driving distance will be reduced when frequently driving on slopes, rough ground or climbing kerbs. The range could be lower than 35 km when non-stop climbing.

**Attention:**

- Your centre of gravity moves when you drive up or down a slope. Please beware.
- Turn around carefully on a steep slope as there is a greater risk of overturning.
- Always mount and leave a pavement in a forward, straight line. Accelerate very gently. It is better to mount the pavement using a curb cut.
- The maximum bump height you can drive over is 100 mm.
- Soft, slippery or uneven surfaces increase the risk of a dangerous situation.

**Transporting the mobility scooter without a driver**  
(see also the chapter 'Transport and storage')

**Life-threatening:**

- When transporting the mobility scooter in a motor vehicle, always make sure you are seated in a fixed seat of the motor vehicle. Mobility scooters may never be used as seating support in a motor vehicle, even if the mobility scooter is secured with a taxi restraint system.

**Warning:**

- Make sure that your fingers do not get trapped when folding down the mobility scooter.
- Never carry the mobility scooter by the plastic parts of the housing.

**Attention:**

- Make sure that the mobility scooter is fully assembled before use.

**Miscellaneous**

**Warning:**


- Do not put the mobility scooter in direct sunlight: the upholstery can become too hot, and cause burns or irritation to the skin.

**Be careful:**

- Do not allow the mobility scooter to come into contact with seawater: seawater is corrosive and will damage the mobility scooter.
- Do not allow the mobility scooter to come into contact with sand: sand may penetrate to the moving parts of the mobility scooter, causing faster wear and tear.

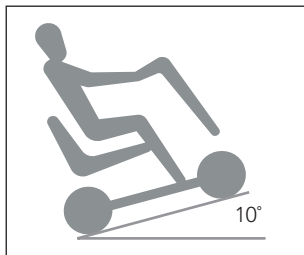Maximum gradient

Approaching a pavement

## Introduction

### Attention:

A number of technical precautions have been made to further guarantee your safety:

- *The immobiliser is activated whenever a battery charger is connected.*
- *The speed in bends is automatically limited.*
- *Free wheeling is limited.*
- *The display warns you if the battery voltage is too low.*
- *The display gives you information for resolving possible faults.*
- *In the event of an electronic fault, the mobility scooter makes a controlled stop.*
- *The mobility scooter has ergonomically shaped safety handlebars to prevent injuries to fingers and hands.*
- *To avoid the risk of parts of the body becoming jammed, the necessary components have been shielded.*
- *Your dealer can fit a seatbelt (anterior pelvic support) to the fastening points of the armrests that are attached to the backrest, to secure your body in the correct position.*

### EMC-approved

The mobility scooter is approved for EMC (Electro-Magnetic Compatibility). Tests have shown that mobility scooters are not sensitive or damaging to mobile communication equipment and electronically controlled equipment. Despite this, please be careful when using the mobility scooter and mobile or electronic equipment at the same time.

### Warning:

- *The driving performance of the scooter can be influenced by electromagnetic fields (e.g. those emitted by cellphones, electricity generators or high power sources, etc.)*
- *The scooter might disturb the operation of devices in its environment that emit electromagnetic fields (e.g. alarm systems of shops, automatic doors, etc.)*

The Presto is an electrically powered mobility scooter, suitable for both young and older people who have difficulty walking long distances.

You can set up the Presto mobility scooter to meet all your requirements. This guarantees a relaxed posture while driving. The mobility scooter is basically operated with two hands. You need not be strong to drive the mobility scooter and the controls are easy. The Presto is available as a 4-wheel scooter.

This manual describes the default settings of your mobility scooter. In the back of this user manual a list has been included for recording your specific settings.

### Intended use

The Presto is intended to provide a means of mobility for people with a reduced walking ability. The product provides a seating support.

In the seated posture the user is applying the drive and steering functions of the product.

The Presto is a configurative product that can be configured according to the pricelist of the product.

## Technical specifications Presto/ Presto S

- Intended use: outdoors, between -25° C and + 50° C, on paved paths and roads
- L x w x h: 1340 x 650 x 1135 mm (Presto)
- L x w x h: 1385 x 650 x 1135 mm (Presto S)
- Height with headrest +180 mm
- Height without seat 690 mm
- Weight without batteries: 87 kg
- Weight with batteries (60 Ah): 122 kg
- Weight of heaviest removable part (seat): 23 kg
- Maximum static stability downhill: 15 °
- Maximum static stability uphill: 15 °
- Maximum static stability sideways uphill: 15 °
- Maximum dynamic stability uphill: 10 °
- Rated slope: 10 °
- Max. driving theoretical distance according to ISO 7176-4 with fully-charged batteries\*:  
29 km (18 miles - 50 Ah) / 35km (22 miles - 60 Ah) / 44 km (27 miles - 73 Ah)
- Maximum obstacle height: 100 mm
- Maximum obstacle drop (pavement): 100 mm
- Ground clearance: 80 mm
- Max. (programmable) speed forward: 15 km/h\*\*
- Minimum braking distance from maximum speed (programmable): 3600 mm
- Seat angle: 3 °
- Seat depth: 440 mm
- Seat width: 510 mm
- Seat height above footrest (min. - max.): 470 - 545 mm
- Height of footrest above ground: 190 mm
- Backrest angle: 90 - 110°
- Backrest height: 500 mm
- Armrest height (to seat): 190 - 240 mm
- Armrest length (to backrest): 350 mm
- Turning circle (radius): 1600 mm
- Turn around width: 2000 mm
- Test dummy weight: 160 kg
- Max. user weight: 160 kg
- Required operating force: approx. 1-30N
- Battery capacity (optional): 50/60\*\*\*/73\*\*\* Ah
- Max. load: 8 amps

\* Depending on the set speed, outside temperature, weight of the user, tyre pressure, etc.

\*\* Less is possible

\*\*\* Gel batteries

- Motor, transaxle: 850 watts 24 V (Presto)
- Motor, transaxle: 1000 watts 24 V (Presto S)
- Charger type: 8 Amp 24 V
- Maximum sound power level 75 Db
- Magnetic brake: 8 Nm 24 V
- Ability to dismantle for transport: limited (See also page 29)
- Wheel size: 3.00 – 8 (Tubeless tire)
- Complies with: EN 12184 class C

The mobility scooter has 2 12-volt dry (gel) batteries.

The following types are suitable:

- 50 Ah (198 x 166 x 171 mm) (14,2 kg)
- 60 Ah\*\*\* (260 x 170 x 178 mm) (17,7 kg)
- 73 Ah\*\*\* (260 x 170 x 210 mm) (23,6 kg)

### Test specifications

- The scooter conforms to the following standards: ISO 7176- 8,9,14,16
- Requirements and test methods for static, impact and fatigue strengths: ISO 7176-8
- Climatic tests for scooters: ISO 7176-9
- Power and control systems for scooters: ISO 7176-14
- Requirements for resistance to ignition of upholstered parts: EN1021-2

### Service and technical support

Contact your dealer if you have any questions about specific settings, maintenance or repairs. Always have the serial number ready. You can find these on the mobility scooter's CE sticker (see also page 20).

### Expected service life

Life & Mobility assigns a service life of 6 years to this product. This means that during this period, the product is fixable, safe and usable.

The statement above is only applicable if the user uses the product in the manner that Life & Mobility intended, marked with CE and including any original accessories.

## Product description Presto


### Checking the delivery

Before using the mobility scooter, check if the delivery matches the following product description. If you find any transport damage upon delivery, please inform your dealer. Life & Mobility trusts that the mobility scooter meets your expectations and hopes it brings you much convenience.

### Product description


Mobility scooter

- A. Handlebars with control panel
- B. Basket
- C. Headlight
- D1. Indicators (orange)
- D2. Brake lights / Rear light (red)
- D3. Day lights (optional)
- E. Front wheels
- F. Footrest
- G. Battery controller/electronics cover
- H. Freewheel lever
- I. Rear wheel
- J. Anti-tippers (optional)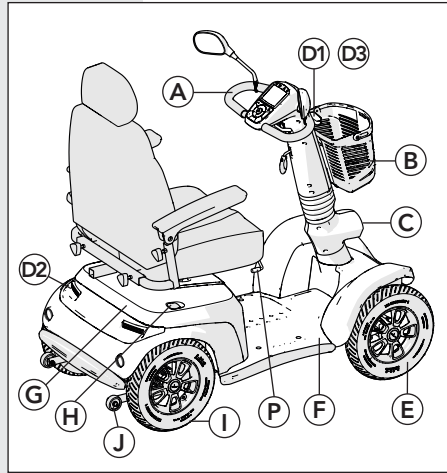


Seat

- K. Seat
- L. Folding armrests
- M. Armrest angle adjustment
- M1. Armrest height adjustment
- M2. Armrest width adjustment
- N. Back angle lever
- O. Seat lever (rotate)
- P. Seat depth lever
- Q. Headrest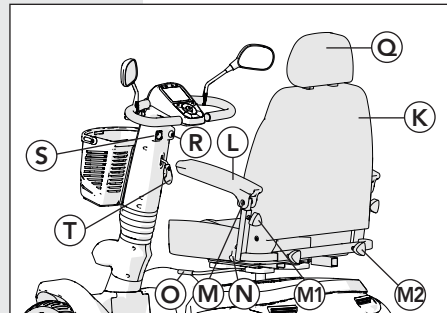

Handlebars

- R. Ignition lock
- S. Battery charger contact
- T. Handlebar angle adjustment lever

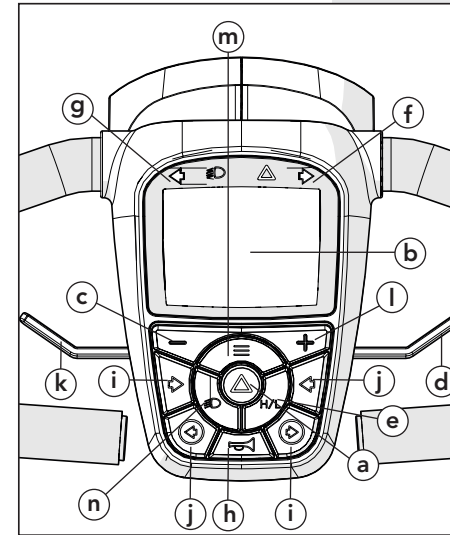Product description control panel

Control panel

- a. Warning lights
- b. Display
- c. Speed decrease button
- d. Forwards / reverse throttle lever\*
- e. Hi Lo function
- f. Right indicator
- g. Left indicator
- h. Horn
- i. Right indicator
- j. Left indicator
- k. Forwards / reverse throttle lever\*
- l. Speed increase button
- m. Menu button
- n. Lights

### Features

- Basket
- Lights
- Brake lights
- Horn
- Slide system under chair
- Torsion springs on front and rear wheels
- Semi-independent suspension on the rear axle and on the front axle
- Comfortable seat with preshaped seating surface and backrest
- Easy and light operation of handlebars and control levers
- Shown on the display: km/h, total distance (km), time, temperature, high/low speed, sound setting
- Clear battery display and low battery voltage warning

\* Depending on how the mobility scooter is set up.

**Safety and comfort features**


- Ergonomic safety handlebars with soft padding and height and depth adjustment via a gas spring
- Rotating headlight
- Limited freewheel (4 km/h)
- Automatic speed limiter in turns
- Indicators cancel automatically after turn
- Mirror
- Adjustable armrests
- Automatic brakes (see also page 20)
- Headrest
- Anterior pelvic support

**Not pictured**

Various accessories are optionally available, including:

- Seatbelt
- Rearview mirror
- Anti-tipping supports
- Eiffel crutch holder
- Rollator holder
- Lined scooter cape, rain and wind proof
- Rainproof scooter cover
- Adapter for Goliath 3 / Hercules tackle system

See [www.life-mobility.com](http://www.life-mobility.com) for more accessories.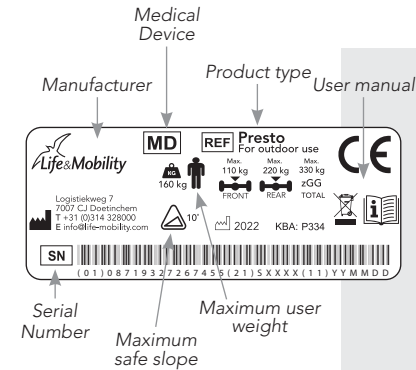


**Markings on the mobility scooter**

There are a number of stickers on the mobility scooter. Below is an overview of these stickers, with their location.

**CE sticker**


The mobility scooter complies with the mandatorily applicable European REGULATION (EU) 2017/745 for Medical Devices and has a CE label.

The CE sticker is located underneath the battery cover (J) at the rear of your mobility scooter. Tilt the cover backwards and place this separately.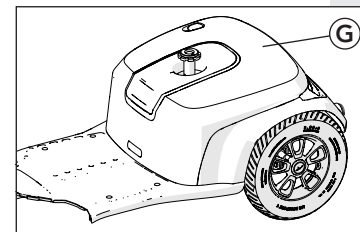

**Attention:**

- Always place back the batterycover (G), as it protects the electronic equipment underneath.
- You will hear a click when the electronics cover is in the correct position.

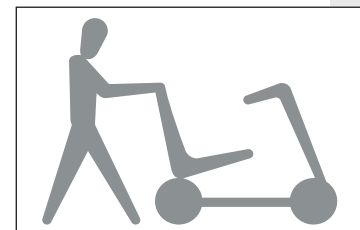Push position

**Freewheel sticker (see also page 29)**

The sticker above the freewheel lever indicates the "push position". In this setting the mobility scooter can be moved freely.

The sticker below the free-wheel lever indicates that the mobility scooter is ready to be electrically driven.

Electric driving position

**Battery connection diagram**

The battery connection diagram is located on the inside of the battery cover. It also includes the fuse and the motor controls.

## Settings and adjustments

You can set up the mobility scooter to meet all your requirements. It is advisable to contact your dealer for the correct settings.

### Adjusting the seat

#### Seat height


The seat is connected to the seat post with a bolt-nut connection and can be adjusted by your dealer

#### Back angle

#### Attention:



- Only adjust the backrest when you are sitting in the seat.
- Watch out for overturning:
  - If you have set up the seat with a large back angle.
  - When driving up a slope.
  - If the slide system is in the rear-most position.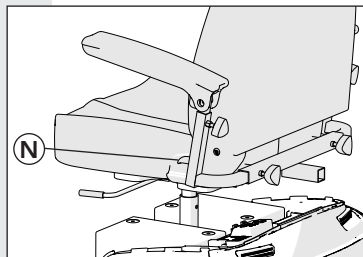

1. Sit down on the seat.
2. Lift up the lever (N).
3. Use your upper body to push the backrest backwards into the right position, or:
4. Bend your upper body forward to allow the backrest to move forward automatically.
5. Release the lever (N).  
The backrest is now locked in its current position.

#### Seat depth

1. Sit in the seat
2. Lift the lever (P).
3. Slide the seat forward or backward until the desired seat depth is reached
4. Release the lever (P).  
The seat is now locked in its current position.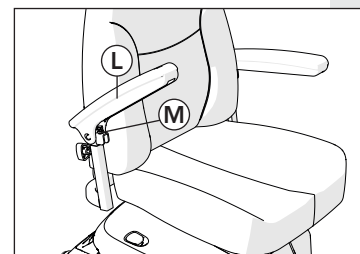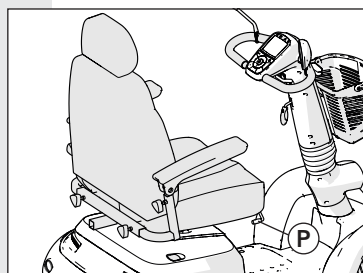

Change armrest angle

#### Armrests


#### Attention:

- The position of the armrests changes if the position of the backrest is adjusted.

#### Armrest angle

1. Rotate the armrest upwards. (L)
2. Release the nut and tighten the bolt (M) to rotate the armrest downward or loosen the bolt to rotate the armrest upward.
3. Fix the nut to secure the bolt.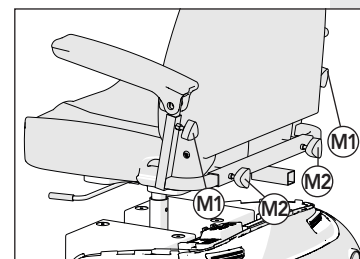

#### Armrest width


1. Release the knob (M2) to adjust the armrest sideways. Fix the knob when correct position is reached.

#### Armrest height

1. Release the knob (M1) to adjust the armrest in height. Fix the knob when correct position is reached.

#### Steering column angle

1. Sit down on the seat.
2. Pull the lever up (T)\*.
3. Pull the steering column towards you or push it away until you reach the desired position.
4. Release the lever.  
The steering column is now locked in place.

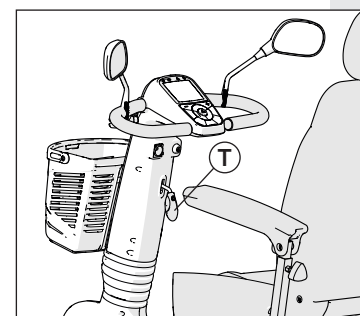Change handlebar angle

\* It is located on the left or right-hand side, depending on the configuration of your mobility scooter.


## Use

### Advance checks

Check the following before driving off:

- Whether the lights are working,
- The battery voltage on the display (see page 28),
- The tyre pressure (see the chapter 'Maintenance'),
- Whether the freewheel lever is pointing down (electric driving position, see page 20),
- Whether the battery charger has been removed.
- Make sure additional loads are only carried in the basket B (maximum load: 3kg) or on the accessory point U (maximum load: 10kg)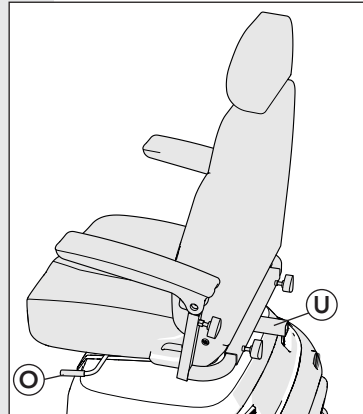

Product description seat


### Getting in and out by yourself

#### Attention:

- Make sure that the seat is properly locked in place before driving off.

#### Getting in/out

1. Push the seat lever (O)\* down.
2. Rotate the seat a quarter turn anticlockwise or clockwise.
3. Get in or out.
4. Rotate the seat back again until you hear a click.  
*The seat is now locked in place.*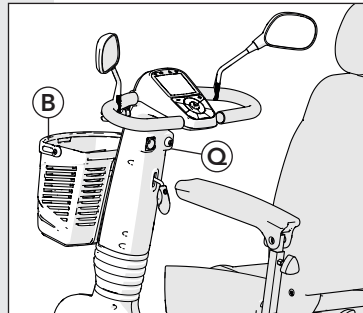

Product description handlebars

#### Starting


Always turn off the scooter when entering or exiting the scooter. When the scooter is turned off, the time is displayed

1. Insert the key in the ignition (O).
2. Turn the key a quarter turn clockwise.
3. Wait until the battery indication appears on the display.



*The mobility scooter is now ready to drive.*

#### Warning:

- Do not use the scooter if it is behaving abnormally or erratically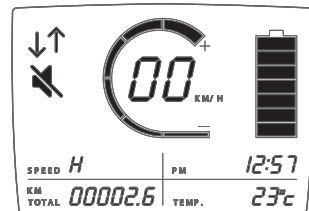

Ready for use

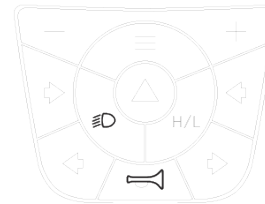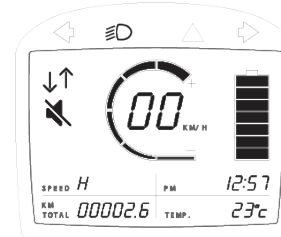- See page 31 for error codes that can be displayed.
- Do not operate the scooter with depleted batteries, since the user can be stranded

### Speed

The speed can be adjusted for either indoor or outdoor use by selecting the high or low function (H/L).

#### Warning:

- Adjust your speed before steering into a bend. The bend speed limiter will not adjust your speed until the bend is too tight. The scooter may overturn.
- Despite the small turning circle of the mobility scooter, it may still be difficult to ride into lifts or doorways. Make sure that you always use the correct low speed setting for tight manoeuvring.

#### Attention:

- Adjust your speed using the throttle lever. The speed settings determine how fast you can go if you squeeze the throttle lever all the way.
- Check that the road behind you is clear when reversing.
- The mobility scooter reverses more slowly than when it drives forward.
- An alarm can be heard when reversing.
- Upon request, your dealer can change the control (method) of the throttle lever.

By default the speedometer is set to 6 blocks, which is 100% of the total power. By default the maximum speed is set to 9 mph (15 km/h). Your dealer can adjust the speed setting for you.

1. Press the + or - button to adjust the speed setting.


- Choose a suitable speed setting for each situation.
- Indoors, in shops, etc., 1 to 2 blocks (preset Low)
  - On the pavement: up to 3 blocks (preset Low)
  - In busy streets: up to 4 blocks (preset High/Low)
  - On quiet streets: up to 6 blocks (preset High/Low)

### Driving forward and reversing

#### Attention:

- The scooter has automatic brakes. When releasing the throttle lever, the brakes are applied automatically.
1. Slowly squeeze the throttle lever (k)\*. The mobility scooter starts moving forwards.
  2. Squeeze the throttle lever further. The mobility scooter will start moving forward more quickly.
  3. Release the throttle lever. The mobility scooter will stop. The brake lights are activated.
  4. Use lever (d)\* to reverse.
  5. In case of an emergency, the scooter can be stopped by switching the key to off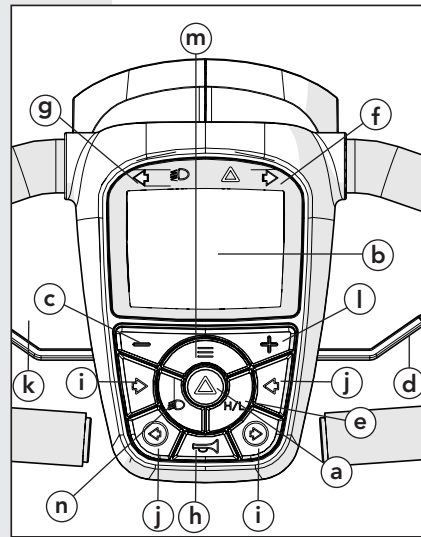

Control panel


### Lights

#### Attention:

- After a tight bend the indicators automatically switch off.
  - Your dealer can disable the default sound from the warning lights.
1. Press the (n) button to switch the lights on or off.
  2. Press the (i) or (j) button to switch the indicators on or off.

\* Depending on how your mobility scooter has been set up.

3. Press the (a) button to switch the warning lights on or off.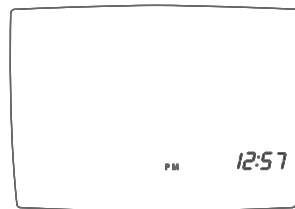

Stand-by activated

### Batteries

#### Caution:

- New batteries must be run in. The full capacity of batteries is only reached after they have been used and charged a couple of times. Take this into account during your first few trips.
- Charge the batteries every day if you drive a lot, and every few days if you drive little. This will extend the service life of the batteries.
- Always charge the batteries to full capacity.
- Avoid draining the batteries completely. The mobility scooter will lose power which will have a negative effect on the service life of the battery.
- Charge fully drained batteries as soon as possible using a 24-hour cycle.
- Do not keep your batteries fully drained for a prolonged period of time.
- Never put metal parts on the batteries: this may short-circuit the batteries and damage them.

#### Attention:






- At lower temperatures the capacity of the batteries is reduced. This also reduces the operating range.
- The operating range is based on the MK batteries delivered by Life & Mobility. Other batteries can have a different range.
- Always connect the charger when putting the mobility scooter away for the night.
- You cannot overcharge the batteries supplied. Thanks to a voltage limiter, the charger automatically switches off when the batteries are full.
- Only use Life & Mobility battery chargers.
- If you use batteries other than the batteries supplied by Life & Mobility ensure that a good retaining ring is fitted to prevent them moving. If a retaining ring is not fitted this could lead to short-circuiting.

### Reading the battery status

The battery-shaped indicator on the display shows the remaining battery capacity:

| Battery status     | Indicator  |
|--------------------|--|
| All blocks visible | The batteries are full.  |
| 3 blocks visible   | You can drive another 5 - 10 km (3 - 6 miles) (depending on the type of battery and other circumstances). You should start charging now. |
| 1 block visible    | Charge the battery of the mobility scooter as soon as possible. The display shows a warning.   |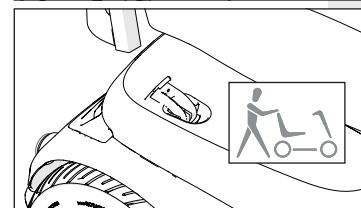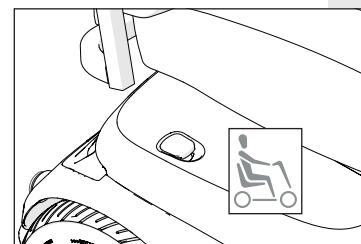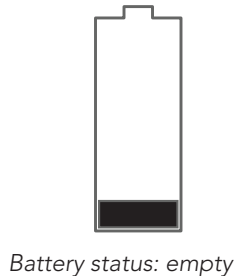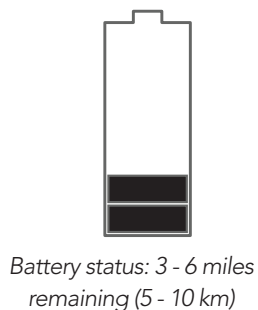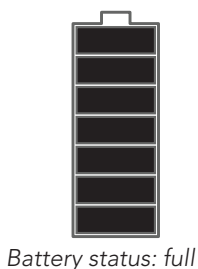

Freewheel lever

### Freewheel lever

#### Warning:

- Never use this function on a slope or while driving. It will disable the brakes. Always push the freewheel lever downwards to re-engage the brakes after using the freewheel function.


- Pull the freewheel lever upwards to move the mobility scooter by hand. A beep sounds every 2 seconds to remind you that the freewheel function is engaged.
- Push the freewheel lever down for the electric drive position. Teach yourself to do each time you have used it.

#### Attention:

- If you accidentally roll down a slope in freewheel mode, the scooter will automatically slow down to a speed of 1 km/h. You can decrease the breaking distance by pushing the freewheel lever down.

### User menu

- Home screen  
By default the (home) screen of the control panel displays the following information:
  - the distance covered (A),
  - the time (B),
  - the battery status (C),
  - the speed (D)
  - the temperature (E)
  - speed setting (F)
  - sound setting (G)
- Press the menu button for 5 seconds.  
The second menu (distances) is displayed and shows the following: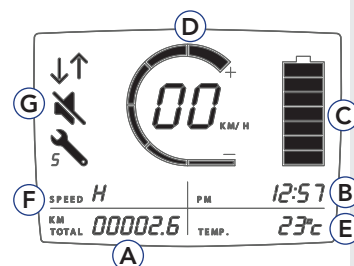

Home screen

### Charging batteries

#### Warning:

- Only use the original charger type that is supplied with your scooter. Other changers can cause damage to the batteries, the electrical wiring and can even cause fire.

- Insert the plug of the battery charger into the battery charger contact (S).
- Insert the plug of the mains cord in a wall socket. The red LED is on when charging, the green LED is on when battery is full.


#### Warning:

- When both LED's are flashing, the polarity is wrong, there is overvoltage or short circuit.
- Always fully complete the charging cycle of 8 - 10 hours. This will prevent battery damage.

### Replacing batteries

#### Warning:

- Batteries should only be replaced by qualified personnel.

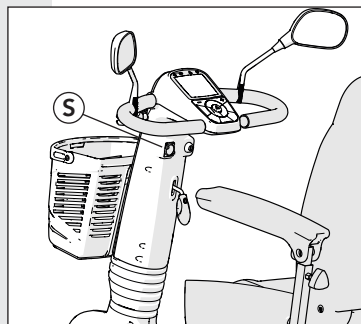Battery charger contact

## Maintenance

### Cleaning

#### Be careful:

- Do not use bleach or solvents.
- Do not use a high-pressure cleaner to clean the mobility scooter.

#### Attention:

- Regularly clean the upholstery and the frame.

#### Upholstery

1. Remove any light dirt with a solution of a mild cleaning agent and warm water.
2. Carefully scrub the upholstery with a soft brush if the dirt has been absorbed into the upholstery.

#### Frame and covers

1. Clean the housing with a non-abrasive cleaning product.
2. Apply non-abrasive car wax to the entire frame and only to the painted covers.



### Batteries (see also the chapter 'Use')

#### Be careful:

- Batteries can be dangerous. Only allow your batteries to be installed and maintained by a qualified scooter mechanic. They have the required qualifications and tools to perform the installation and maintenance safely and correctly.
- The scooter uses two batteries with a 24-volt series connection. Never use both batteries at the same time:
  - Batteries from different manufacturers
  - Batteries with different technologies
  - Batteries with different date codes

- The time setting (B)
3. Press the menu button again.  
*The third menu is displayed and shows the following:*
    - The temperature setting (E)
  4. Press the menu button again.  
*The fourth menu is displayed and shows the following:*
    - Distance setting (A)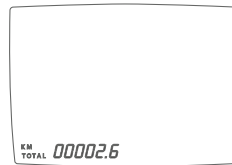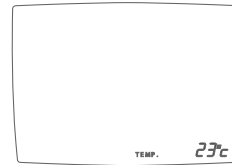

Setting the time

### Setting the time, date, temperature and distance value

1. Turn on the scooter
2. Press "menu" button for 3 seconds
3. Press "+" to switch between Am and PM
4. Press "menu" to set hour with "+" or "-" button
5. Press "menu" to set minutes with "+" or "-" button
6. Press "menu" to set temperature setting
7. Press "+" or "-" to switch between °C and F
8. Press "menu" to set driven distance value
9. Press "+" or "-" to switch between km and miles
10. Press "menu" to return to main screen

### Parking

When parking your scooter, place a chain lock through one of the wheels for added security.

**Attention:**

- Always fully charge batteries before using them for the first time.
- Charge the batteries continuously for 24 hours at least once a month.
- The battery connection method is illustrated on a diagram on the inside of the battery cover.
- Be careful with damaged batteries. Leave (environmentally-friendly) replacement of the batteries to your dealer.

**Tyres****Warning:**

- For safety reasons always ensure that the tyre pressure on your mobility scooter is correct. If the tyre pressure is too low, your scooter's roadholding will deteriorate, its operating range will decrease and the steering will become heavier.

**Attention:**

- The Presto has tubeless tires.
- In the event of a flat tyre, please contact your dealer. However it is possible to drive at very low speed and short distance with a flat tyre it is recommended to minimize.
- The pneumatic tyres of the mobility scooter must have sufficient pressure. Check this regularly.
- If the tyre pressure is too low you will not be able to drive as far.
- The pressure of the front tyre must be at least 3 bar.
- The pressure of the rear tyres must be at least 3 bar.
- The maximum tyre pressure is 3.5 bar.
- If the tyre pressure is too high, driving will become uncomfortable.
- Regularly check the tyres for wear and tear (due to drying out).

**Service and technical support**

At least once a year have the mobility scooter checked and serviced by your dealer. Preventive maintenance ensures that the proper and correct functioning of all moving parts is checked, as well as the electronics and the condition of the batteries.

Repairs should only be performed by your dealer or by Life & Mobility, and only original parts supplied by Life & Mobility should be used for this.

For information about product safety notices and recalls contact Life & Mobility.


## Transport and storage

### Maintenance matrix

The following matrix indicates which parts should be regularly checked. If a defect occurs, contact your dealer to resolve this issue.

| Part               | Action   | Daily | Weekly | Monthly | Each quarter | Annually |
|--------------------|--|-------|--------|---------|--------------|----------|
| Battery charger    | Check functioning                                      | X     |        |         |              |          |
| Tyres              | Check tyre pressure (front and rear : 3 - 3.5 bar)     |       | X      |         |              |          |
| Lights             | Check functioning                                      | X     |        |         |              |          |
| Housing            | Clean with damp cloth                                  |       | X      |         |              |          |
| Connections        | Check for breakages                                    |       | X      |         |              |          |
| Rotating mechanism | Check functioning and play                             |       |        | X       |              |          |
| Batteries          | 24-hour charging cycle                                 |       |        | X       |              |          |
| Parking brake      | Check functioning                                      |       |        |         | X            |          |
| Tyres              | Check wear and tear                                    |       |        |         | X            |          |
| Mobility scooter   | Servicing  |       |        |         |              | X        |
| Potentiometer      | Check the potentiometer for play in the zero position* |       |        |         |              | X        |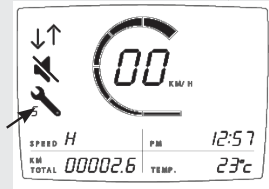

\* By moving the throttle lever, you can clearly feel any play in the zero position. Unusual instability of the throttle in the zero position deserves extra attention (manufacturer's advice) or immediate replacement of the potentiometer.

### Instructions for receipt, redeployment, and ownership transfer of mobility scooters

When receiving a mobility scooter with the intention of delivering it as new, second-hand, or for redeployment, check all components listed in the maintenance matrix, with the exception of the annual potentiometer check and the routine service inspection.

### Transporting the mobility scooter

#### Extremely dangerous:

- When transporting the mobility scooter in a motor vehicle, always make sure you are seated in a fixed seat of the motor vehicle. Mobility scooters may never be used as a seat in a motor vehicle, even if the mobility scooter is secured with a taxi restraint system.
- Fixate the scooter into the car by applying straps trough all 4 rims to the fixation points on the floor of the car.
- An unoccupied scooter can be transported by air, boat or road. Ask the transporter about special safety precautions during transport

#### Be careful:


- Never carry the mobility scooter by the plastic parts of the housing.

#### Removable parts (without the use of tools)

- Basket
- Seat.
- Headrest.
- Armrests

#### Folding down the backrest

- Stand next to the mobility scooter.
- Firmly grip the backrest.
- Lift lever (N).
- Push the backrest forward.
- Release lever (N).

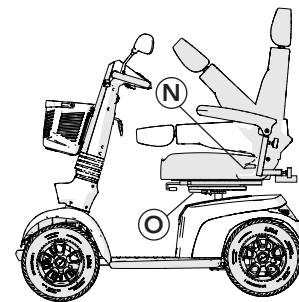Folding down the backrest

#### Removing the seat

1. Turn lever (O) downwards.
2. Turn the seat a little.
3. Lift the seat from the mobility scooter.  
Place one hand at position (O) and the other hand behind the backrest to lift the seat

#### Attention:


- The seat is heavy, > 20 kg. Ask for help if required.

#### Folding down the steering column

1. Stand next to the mobility scooter.
2. Pull the handlebar adjustment lever up (T).
3. Move the steering column as far as possible towards the seat.

*Tip: when the seat is removed, the steering column can be folded down further.*

4. Release the lever.

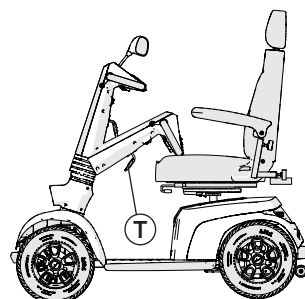Folding down the steering column

#### Transport

1. Pull the freewheel lever upwards.
2. Push the mobility scooter into the vehicle.
3. Switch off the mobility scooter.
4. Please ensure that the mobility scooter is properly secured.

#### Storage when not using the mobility scooter

#### Attention:

- Always store batteries fully charged.
- Fully charged gel batteries and AGM batteries will retain their charge for up to six months, but they must always be charged before use.
- Avoid storage of batteries at extremely high or low temperatures.
- Sealed gel batteries and AGM batteries do not need to be topped up with water. Contact a qualified scooter mechanic if you have any performance issues.

#### Longer than a week:

- Connect the battery charger at least once a week to maintain the condition of the batteries or put the mobility scooter away for the night with the battery charger connected.

#### Very long term (over 4 months) (e.g. during winter or due to illness):

- Have your dealer disconnect the wiring of the batteries.

## Faults

## Warranty

### General

#### Attention:

- Never attempt to resolve a fault yourself. Contact your dealer instead.

### Error codes

1. An abnormal condition may be indicated by a flash code on the display.
2. Depending on the condition, the scooter may or may not allow driving. In some cases driving may be allowed but in a reduced speed („limp“) mode.
3. See table below for error codes, the meaning and way to solve the issue.
4. Contact your dealer to repair the error if resetting does not help.
5. Pass the error message on to your dealer.

| Flash | Description                                  | Meaning   |
|-------|--|---|
| 1     | Battery Low                                  | The batteries are running low or are in an overdischarged state.<br>• Recharge the batteries.   |
| 2     | Low Battery Fault                            | The batteries have run out of charge.<br>• Recharge the batteries.<br>• Check the battery and associated connections and wiring.  |
| 3     | High Battery Fault                           | Battery voltage is too high. This may occur if overcharged &/or travelling down a long slope.<br>• If travelling down a slope, reduce your speed to minimise the amount of regenerative charging.   |
| 4     | Current Limit Time-out or Controller too hot | The motor has been exceeding its maximum current rating for too long.<br>• The scooter may have stalled. Turn the controller off, leave for a few minutes and turn back on again.<br>• The motor may be faulty. Check the moto and associated connections and wiring.                               |
| 5     | Park Brake Fault                             | Either a park brake release switch is active or the park brake is faulty.<br>• Check the park brake and associated connections and wiring.<br>• Ensure any associated switches are in their correct positions.  |
| 6     | Drive Inhibit                                | Either a Stop function is active or a Charger Inhibit or OONAPU condition has occurred.<br>• Release the Stop condition (seat raised etc.)<br>• Disconnect the Battery Charger<br>• Ensure the throttle is in neutral when turning the controller on.<br>• The Throttle may require re-calibration. |
| 7     | Speed Pot Fault                              | The throttle, speed limit pot, SRW or their associated wiring may be faulty.<br>• Check the throttle and speed pot and associated connections and wiring.   |
| 8     | Motor Voltage Fault                          | The motor or its associated wiring is faulty.<br>• Check the motor and associated connections and wiring.   |
| 9     | Other error                                  | The controller may have an internal fault.<br>• Check all connections and wiring.   |

### Terms and Conditions of Warranty

The Life & Mobility products are made with care and are carefully inspected before leaving the factory. Should it emerge that a product does not meet expectations please contact the party where you purchased the product. Life & Mobility respects the laws and regulations that protect the consumer and that apply in the country in which the product is purchased.

The warranty does not apply to defects caused by: failure to observe the instructions for use and maintenance, use other than normal use, wear and tear, negligence, overloading, third-party accidents, non-original parts used and defects not caused by the product.

This warranty replaces any other warranty, either stipulated by law or communicated orally, except for anything guaranteed in writing by Life & Mobility.

Warranties are only valid within the EU.

## Disposal of product and packaging

### **Disposal of product and packaging**

All parts of your mobility scooter are recyclable. Your dealer will know the collection points for disposal of the mobility scooter.

From an environmental perspective, we recommend that you dispose of the packaging and the mobility scooter by separating the waste as much as possible, allowing the waste materials to be recycled efficiently.

*This manual has been prepared with the utmost care and is based on information known to Life & Mobility at the time of publication.*

*The information in this manual applies to the standard version of the product. Life & Mobility does not accept responsibility for any damage that may occur due to changes or modifications to the product after selling it for the first time. Neither can Life & Mobility be held liable for any damage caused by third-party work.*

*The information in this manual may not be reproduced for commercial goals and/or made public by any means without prior written permission from Life & Mobility.*

*Pursuant to the Trade Names Act, the trade name and trademarks used by Life & Mobility may not be considered free.*

*No rights can be derived from this text. Subject to changes.*

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*© Life & Mobility – All rights reserved*