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
Implementation

The Remote Stop includes the following main components:

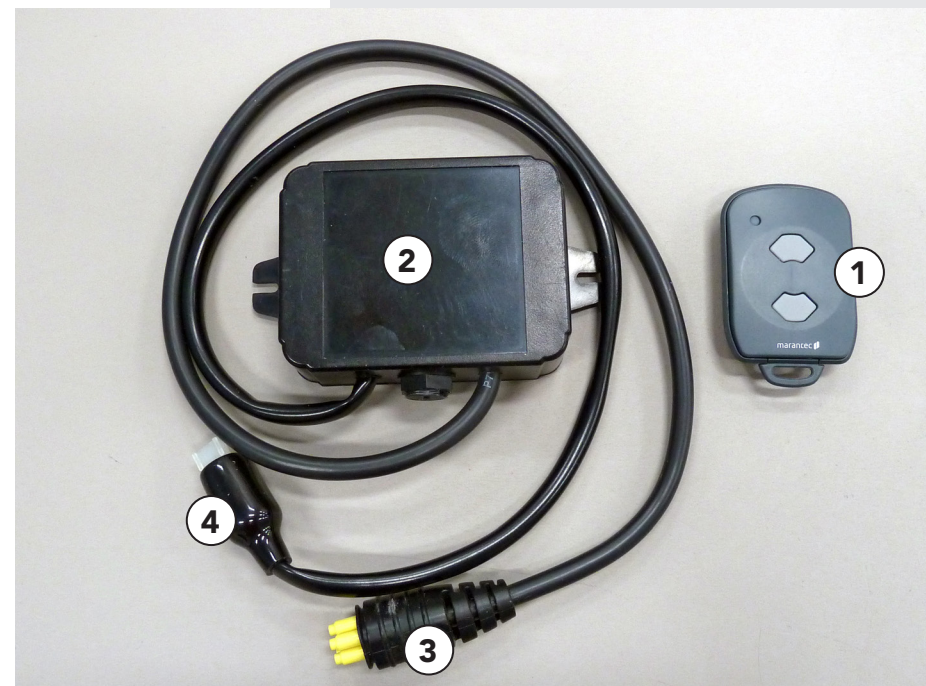1. Hand-held transmitter: Marantec Digital 392 with 2 channels. 868 MHz version
2. Receiver: Marantec Digital 339.2 868 MHz
- 2a Fuse: 250V, 250mAT, 20x5mm (in housing)
- 3 Curtiss Wright R-Net bus cable for power supply
- 4 Curtiss Wright R-Net Inhibit cable for blocking the wheelchair.

Function

The Remote Stop Module is a functional addition to an electric wheelchair that can be used to switch off the wheelchair's joystick function remotely (within a certain distance) using a hand-held transmitter. If the wheelchair is moving when the joystick function is switched off, the wheelchair brakes and comes to a standstill. This braking distance corresponds with the braking distance expected by the wheelchair user.

After the wheelchair controls have been blocked, they can be unblocked using the hand-held transmitter (see Operation).

The blockage cannot be cancelled by switching the wheelchair controller on and off. It can only be done using the hand-held transmitter.

WARNING

The Remote Stop Module is not a safety mechanism and may therefore not be used as an emergency stop device.

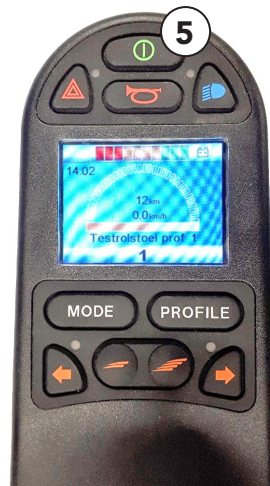
If the safety of persons depends on blocking the driving function, besides the Remote Stop Module there should also be another way of stopping the wheelchair in order to intervene on time.

Use

The Remote Stop module consists of a hand-held transmitter and a receiver. The hand-held transmitter is supplied in a separate package with the wheelchair. When using the hand-held transmitter for the first time:

- Remove the transmitter from the packaging
- Check that the wheelchair batteries are charged (see the enclosed Wheelchair User Guide for instructions)
- Switch on the wheelchair using the ON button on the wheelchair controller* (5)

* The display depends on the chosen wheelchair joystick module

- The hand-held transmitter is equipped with two identical buttons (6). The buttons have the same functionality.
- Test the hand-held transmitter by pressing the control button (6).
- An icon of a red tortoise (7) then appears on the display of the wheelchair controller.
- The wheelchair is now blocked. The wheelchair user can no longer operate the wheelchair using the joystick.
- You can unblock the wheelchair by pressing the control button (6) again.
- The icon of the red tortoise has now disappeared from the display of the wheelchair controller.
- The wheelchair user can now use the wheelchair again.
- After you have followed these instructions, the system is ready for use.

Operation

The Remote Stop module consists of a hand-held transmitter and a receiver built into the wheelchair. You can use the transmitter to block the operation of the wheelchair and then unblock it again. You can only unblock the wheelchair with the hand-held transmitter. The remote control is fitted with a control button (6) that you can use to switch the wheelchair on and off.

Caution!

Remember that there may be a time delay between operating the hand-held transmitter and blocking the operation of the electric wheelchair.

Power supply

The receiver built into the wheelchair is connected to the wheelchair's main batteries. The Remote Stop increases the consumption of standby power. You should charge the batteries at least once a month. By regularly charging the wheelchair batteries, you ensure that the receiver is always supplied with enough power.

Advice

The receiver will not work if the wheelchair battery is empty. In that case, you should recharge the wheelchair battery with the supplied wheelchair battery charger.

The hand-held transmitter runs on a CR 2032 battery. If the hand-held transmitter is not working (properly), it means that the battery is empty and needs to be replaced.

- Open the back of the hand-held transmitter
- Slide the CR 2032 battery from under the holder
- Insert a charged CR 2032 battery and close the housing

Empty batteries must be handed in at a special battery recycling collection point. Do not combine different types of batteries. Do not combine old and new batteries.

Transmitter range

The transmission range of the Remote Stop depends on the operating environment. Before you start using the Remote Stop, test the distance at which the wheelchair can be switched off. Remember that the range can be less indoors than outdoors.

Storage

If the electric wheelchair with the Remote Stop Module is not being used for a longer period (but less than 1 month):

- Make sure that the wheelchair hasn't been switched off with the Remote Stop Module
- Check the display of the wheelchair controller to make sure that the wheelchair has been unblocked. If it has been unblocked, the red tortoise icon will not be displayed.
- On the wheelchair controller, press the ON/OFF button to switch off the wheelchair. The display will switch off.

Advice

When first using an electric wheelchair that has been stored for a long period, you may need to recharge the wheelchair batteries. Use the supplied wheelchair battery charger to do this.

If you want to store the wheelchair for longer than 1 month, we recommend that you disconnect the battery from the wheelchair. For instructions on how to do this, see the wheelchair user guide.

Programming the hand-held transmitter

To connect a new hand-held transmitter to an existing receiver, the following steps need to be taken. You should ask an authorised dealer to do this for you.

Storing the transmitter code:

- Remove the lid of the receiver housing (this may be built into the wheelchair)
- Programme the hand-held transmitter code on Channel 1
- Press the CH1 button for approximately 3 seconds until the LED starts blinking
- Press the desired button on the hand-held transmitter within 10 seconds
- After the receiver has stored the transmitter code, the LED stays on until you release the transmitter button
- Repeat the previous 3 steps for the second button on the handheld transmitter so that it has the same function as the first button
- Put the lid back on the housing

Malfunctions

Description

The wheelchair is no longer reacting to the hand-held transmitter

Possible cause

Distance between wheelchair and hand-held transmitter too far

The hand-held transmitter signal is being disrupted by external influences

The battery of the hand-held transmitter is too weak

The wheelchair battery is too weak and the receiver is not receiving a signal.

Replace the receiver fuse

There is a malfunction between the receiver and the hand-held transmitter

The wheelchair won't switch on after being stored for a long period

The wheelchair batteries are empty

What you should do

Reduce the distance to the wheelchair and check the functions again

Test the hand-held transmitter in an interference-free environment

Replace the battery of the hand-held transmitter. (See Power Supply)

Charge the wheelchair batteries using the wheelchair battery charger

Specialist wheelchair knowledge is required to change the fuse. Contact your dealer

Contact your dealer

Charge the wheelchair batteries using the wheelchair battery charger

When storing the wheelchair for a longer period (more than 1 month), we recommend that you disconnect the wheelchair battery. Contact your dealer

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