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CONTENTS

	Page
1. Checking on delivery	18
2. Description of the Kelvin^{Comfort}	18
3. Maintaining your Kelvin^{Comfort}	27
4. Transport	28
5. Garantie	30
Checklist	31

1. CHECKING ON DELIVERY

Check whether the modular seat element delivered corresponds to the packing note. If your modular seat element is incomplete or damaged on delivery, please contact your dealer. Your dealer must notify the manufacturer or supplier of the incomplete delivery or damage within two weeks.

2. DESCRIPTION OF THE KELVIN^{Comfort}

The Kelvin^{Comfort} seat element is a patented product and complies with the Essential Requirements for Medical Devices Directive 93/42/EEC. The Kelvin^{Comfort} seat element has a CE mark. The Kelvin^{Comfort} seat element, in combination with the Life & Mobility Canto Nxt (Aura), has been successfully tested by TNO to be a "crash-proof wheelchair" in compliance with ISO 7176-19.

2.1 FEATURES

The Kelvin^{Comfort} is a modular seat element with a dynamic design. Its stable frame has a completely freely suspended cover which adapts to the basic contours of the body. The cover is made up of unzippable cushions with a granule and latex filling. Because the granules can move freely and the latex is soft and springy, the seat is dynamic whilst always enabling the user to sit comfortably even with many changes of posture.

2.2 TARGET GROUP

The target group for the Kelvin^{Comfort} has grown dramatically over the years. The Kelvin^{Comfort} is ideal for passive wheelchair users, for whom posture adjustments are not a top priority. This target group benefits from a comfortable, secure feeling without compromising

on mobility, so that interaction with the environment is ensured. The Kelvin^{Comfort} is a modular seat element which enables the problems associated with sitting to be approached individually. Characteristic of this target group are the functional restrictions and curvature resulting from different neurological and other conditions. By adjusting the form and the amount of padding in the cushions, and the possibility of placing optional support cushions for extra individual support in the canvas covers at the back, an individual shape can be created for appropriate support. The Kelvin^{Comfort} offers this target group maximum support while allowing the user to sit comfortably and in a relaxed position.


2.3 DIMENSIONS

Size	S40	S48	M40	M48	M56	L48	L56
Seat with	34-40 cm	34-40 cm	40-46 cm	40-46 cm	40-46 cm	46-52 cm	46-52 cm
Seat depth	40-48 cm	48-56 cm	40-48 cm	48-56 cm	56-64 cm	48-56 cm	56-64 cm
Backrest height	55 cm	55 cm	55 cm	55 cm	55 cm	55 cm	55 cm

BECAUSE OF THE FLEXIBLE NATURE OF THE KELVIN^{COMFORT}, ALL DIMENSIONS ARE APPROXIMATE.

2.4 CUSHIONS

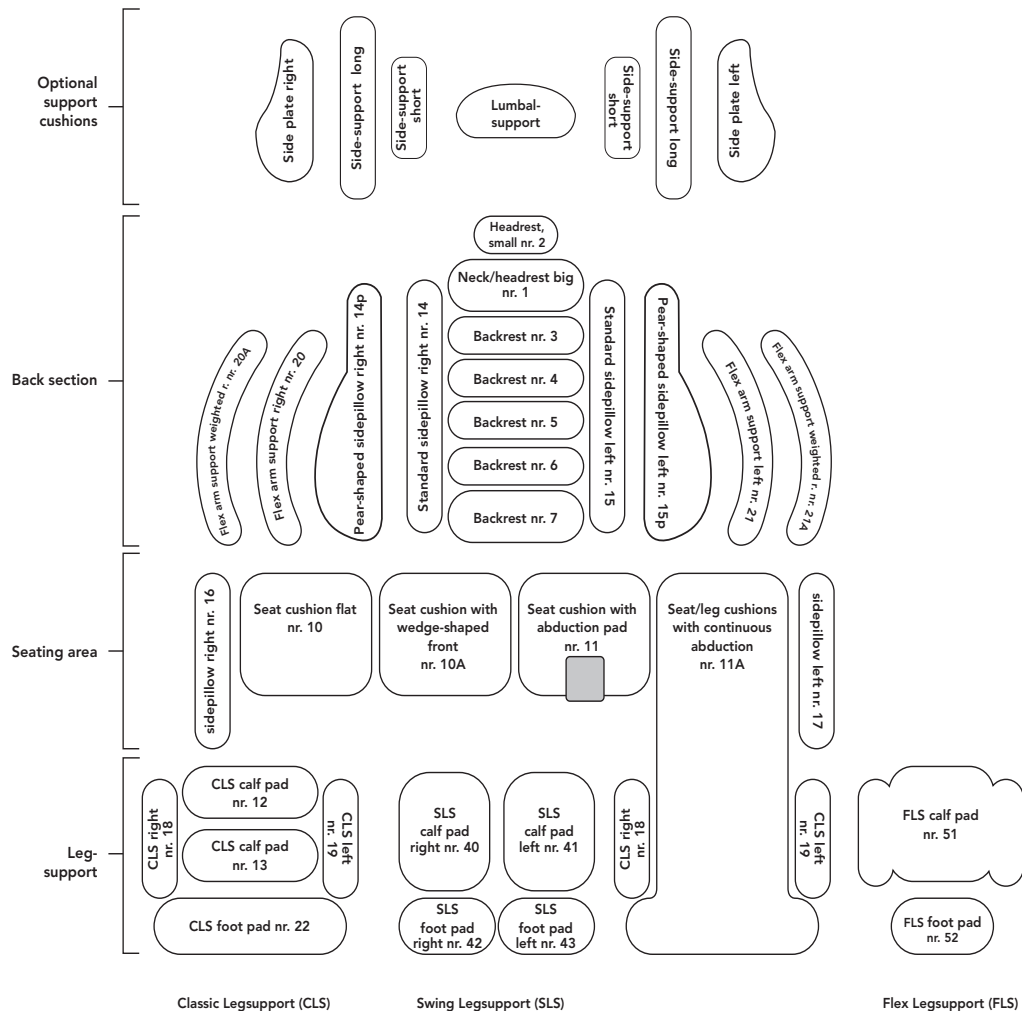The Kelvin^{Comfort} is constructed from various cushions which are fixed to the canvas outside cover of the Kelvin seat element by means of a zipper. All cushions except the seat cushion are filled with a combination of granules (2.5 mm) and latex strings. Each cushion has a different number which corresponds to the number on the cover of the seat system. The cushion arrangements and options can be seen on the next page.

Backrest side cushions

For the backrest side cushions, there is a choice of standard or pear-shaped side cushions, which enables the pear-shaped body of the user to be supported.

Support cushions for additional individual support

Additional optional support cushions can be placed from the back of the canvas cover, by opening the zipper. These cushions can be positioned at the correct individual location while the user is sitting in the chair:

- lumbar support cushions for extra individual support at the height of the lumbar spine
- short side-support cushions for extra local sideways support (this cushion is always used in combination with the plastic side plates)
- long side-support cushions for extra local sideways support over the entire length of the body (this cushion is always used in combination with the plastic side plates)

Seat cushion

The Kelvin^{Comfort} seat cushion is made up of two components: a flexible, soft layer of latex strings and a durable, pressure-distributing foam part. This combination ensures optimum sitting comfort. The seat cushion is fitted with an incontinence cover made of waterproof material.

■ **Flat seat cushion (no.10)**

This is the standard seat cushion. The seat cushion is secured with a zip along the back edge and two Velcro strips underneath.

■ **Seat cushion with abduction pad (no.11)**

If required, a seat cushion with an abduction pad is also available. The abduction pad is positioned between the knees and prevents them from touching. The seat cushion is attached in the same way as the standard seat cushion.

■ Seat cushion with wedge-shaped front 8 cm high (no. 10A)

The seat cushion is higher at the front, so that the user can more easily assume a stable position.

Just like the flat seat cushion, the padding is made of a foam foundation with a comfortable pressure-distributing layer of latex strings.

■ Seat cushion with continuous abduction incline (no. 11A)

The abduction 'hill' runs from the front side of the upper legs to on top of the footrest. As a result of this, the user is prevented from developing pressure points at the spots which carry an increased risk for decubitus. It is divided into different compartments, and the amount of padding can be adjusted according to the wishes of the user.

2.5 LEG SUPPORT

It is possible to install five different types of leg supports on the Kelvin^{Comfort}.

■ Classic leg support (CLS)

This is the standard leg support provided with cushions filled with granule and latex.

The knee angle and the ankle angle can be adjusted by means of toothed hinges. The length of the leg support can be adjusted to the user's calf length. The CLS can also be produced with the seat leg cushion with continuous abduction 'hill'.

■ Flex leg support (FLS)

Because of the slender form of the Flex leg support, the knee angle can be adjusted to less than 90 degrees. The Flex leg support has comfortable cushions which are filled with a granule and latex filling.

■ Swing leg support (SLS)

This is constructed from two separately adjustable, swivelling, and removable leg supports for the left and the right legs. Each leg support can be independently adjusted for knee angle, ankle angle and lower leg length.

■ Comfort leg support

The Kelvin Comfort can also be provided with the Matrixx Comfort leg supports

■ Central leg support,

This Matrixx leg support, too, can be installed on the Kelvin Comfort

2.6 ARMREST

■ Flex arm support

The Flex arm support is secured to the seat element with a zip at shoulder height. The Flex arm support has a granule and latex filling and is also available in a heavier version.

■ Wide armrests

The wide armrests are fitted with insertion tubes which are inserted into the tubes on the seat element.

2.7 HEADRESTS

Depending on the position of the head, you can choose between four different types of headrests. All cushions on Kelvin head supports have a granule and latex filling.

■ Kelvin headrest, small

This is a flexible headrest with a cushion.

■ Kelvin headrest, small, with Edison lever

This is a flexible headrest which can be very easily fixed in many different positions. With a cushion filled with granule and latex. It has a granule cushion. It is used for users who sit off-balance, i.e. whose head is not in a central position or needs more support from the side.

■ Kelvin headrest, large

The width of the headrest matches the width of the seat element. Has a granule and latex cushion. Designed for users who sit 'actively' and whose head needs support in several different positions.

■ Kelvin neck/head cushion

A cushion to support the head which is zipped on to the top of the seat element. For use by smaller users whose head does not project above the backrest.

THERE IS A PLASTIC HEADREST ATTACHMENT ON THE SEAT ELEMENT FOR FITTING THE HEADREST.

2.8 COVERS

There are different covers available for the Kelvin^{Comfort} seat element.

- Terry cover with openings and incontinence material for the seat element. The terry cover is made of 80% cotton for added absorbency.
- Terry cover for the Flex arm support and headrests.

Attaching the covers to the seat element

There are two tapes on the inside of the cover which are threaded through the seat to the outside in the two corners of the seat. There are also two stretch tapes on the edge of the cover along the seat surface which go round the outside instead of through the seat. All tapes must be tied to the tiltable top part of the frame.

Attaching the covers to the Flex arm support

Once the terry covers have been fitted over the Flex arm support, you can attach them to the back tube of the Canto Nxt.

2.9 ADJUSTMENT OPTIONS

■ Backrest adjustment

The KelvinComfort has an adjustable backrest. To adjust the angle of the backrest, you can make use of the special openings in the canvas at the height of the backrest angle hinges to loosen and tighten the toothed hinges using a size 8 socket wrench. The backrest assembly points of the Kelvin^{Comfort} and the backrest angle hinges (and any seating depth configuration) of the Canto Nxt must also be loosened for the backrest angle setting.

If a large backrest angle is selected, the Kelvin Comfort can be fitted with an optional fill cushion that can be set in the space which develops with a large backrest – between the seat cushion and the lower back cushion of the canvas cover – so that the user can be optimally supported.

N.B.

- ONLY ADJUST THE BACKREST WHEN THERE IS NO-ONE SITTING IN THE CHAIR.
- MAKE SURE YOU DO NOT COMPLETELY REMOVE THE BOLT IN THE TOOTHED HINGE.
- WHEN ADJUSTING THE BACKREST, IT IS IMPORTANT THAT AVAILABLE SPACE REMAINS BETWEEN THE KELVIN SEAT ELEMENT AND THE FRAME OF THE CANTO NXT WHEN UNDER LOAD..

3. MAINTAINING THE Kelvin^{COMFORT}

3.1 CLEANING THE CUSHIONS

The cushions are easy to clean with a damp cloth and an all-purpose cleaner that does not contain any aggressive substances and that is suitable for textiles and plastics.

3.2 WASHING THE COVERS

- The terry cover and the Comfort cover for the seat element can be washed at up to 60°.
- The terry covers for the headrest and the Flex arm support can be washed at up to 60°.
- The Flex arm support (without padding) can be washed at up to 60°.

Tip: Use liquid laundry detergent.

- THE FLEX ARM SUPPORTS MUST BE COMPLETELY EMPTIED.
- READ THE LAUNDRY LABEL ON THE COVERS FOR CARE INSTRUCTIONS.

3.3 CUSHIONS

- All cushions except the seat cushion are filled with a combination of granules (2.5 mm) and latex strings.
- Filling level: The cushions are filled to a specific standard. No liability can be accepted if their contents are changed by third parties without consulting the supplier or manufacturer.
- Zips: The zips on the cushions must be completely closed.

4. TRANSPORTING

The Kelvin^{Comfort} seat element, in combination with the Life & Mobility Canto Nxt (Aura), has been successfully tested by TNO to be a “crash-proof wheelchair” in compliance with ISO 7176-19.

However, whether the wheelchair user can be safely transported depends on the state of his or her health; the user’s physician should be consulted. However, Life & Mobility recommends that the user be transferred to a fixed seat if possible.

By ensuring the following conditions, users of the Kelvin^{Comfort} seat element can be safely transported in a suitable taxibus:

- Maximum weight of user = 125 kg.
- The Kelvin^{Comfort} seat element must be equipped with Life & Mobility Matrixx headrests.
- The user must be transported facing forward.
- If a worktop is fitted, it must be removed in order to prevent injury to other passengers. The worktop should be safely stowed away elsewhere in the vehicle.
- If possible, any accessories must be secured or removed.
- The user must use the two-part three-point safety belt of the taxibus. This two-part three-point safety belt must be attached to the taxibus and is intended to prevent injury to the head and chest.
 - The lap belt must be fed through the existing slits in the seat element. The slits can be found at the corners of the seat element. If a terry towelling or comfort cover is used, the lap belt must also be first fed through the existing slits of this cover.
 - The lap belt must be positioned low across the user’s waist. The angle between the belt and the floor must be between 30° and 75° (a greater angle is better).
 - The lap belt must run over the front of the body following the line where the hip and the pelvis meet. The belt must closely follow the body here. The belt must fully support the body.
 - The diagonal belt must run over the seat element, the shoulder and the chest.

- The belt must be sufficiently tight but not be uncomfortable for the user
- The belt must not be twisted.
- The belt must not come in contact with components of the wheelchair (such as armrests or wheels) in such a way that it does not fully support the user’s body.
- Note: The optional wheelchair seat belt is not suitable for use as a safety belt. The seat belt is intended to keep the user in position.
- The Canto Nxt must be secured to the floor of the taxibus using a four-point system. Note: if there is no ex factory hook symbol sticker on the wheelchair, it is not a wheelchair which can be transported safely.
- The Canto Nxt must be secured to the floor using an anchoring system that is suitable for use with the hook eyes.
- If possible, the Canto Nxt should not be transported in a tilted-back position.

The Kelvin^{Comfort} seat element, in combination with the Life & Mobility Canto Nxt (Aura), has been dynamically tested in the forwards direction, where the test dummy (a dummy weighing 75 kg) was restrained with a lap and shoulder belt. Its suitability in other collisions (e.g. lateral) has not been tested.

Following an accident, a representative of the factory must inspect the wheelchair to determine whether it can still be used.

Changes to the fixing points or frame must not be made without first consulting the manufacturer.

5. GUARANTEE

The manufacturer guarantees the frame of the Kelvin^{Comfort} against frame breakage for a period of one year. The manufacturer also guarantees the quality of the sewn seams on the outer cover and the granule cushions for one year.

The guarantee covers manufacturing defects only and excludes: abnormal or improper use, negligence, overloading or accidents. This guarantee supersedes any other guarantee laid down in law or verbally notified, with the exception of any written guarantee provided by the manufacturer.

Guarantees are only valid within Europe.

Defects must be reported to the manufacturer or supplier within 4 weeks.

Please contact your dealer if components need replacing.

Checklist

✓ SITTING POSITION

Seat the user as far back in the seat element as possible so as to prevent them from slumping back in the chair.

✓ BACKREST ANGLE

The wrong backrest angle can affect the sliding force which can result in an increased risk of decubitus. N.B.: Always adjust the backrest angle when there is no-one sitting in the chair. There must be a gap of at least 10 cm between the Kelvin^{Comfort} seat element and the frame.

✓ TILTING

The seat angle can be altered using the tilt adjustment on the frame.

✓ TRANSFER

Transfer the user in and out of the chair with a passive lift; position the user on the tilted seat element from the side.

✓ LIFT MAT

Make sure there are no folds, edges or fastening clips of the lift mat between the user and the seat element as they can increase the risk of decubitus. Remove the lift mat after the transfer if possible.

✓ COVERS

The covers must be positioned in such a way that the laundry label is on the top. Secure the tapes to the tiltable part of the frame. When fitting the covers, make sure they are as smooth as possible with no folds.

✓ CLEANING

The cushions are easy to clean with a damp cloth and an all-purpose cleaner that does not contain any aggressive substances and that is suitable for textiles and plastics.

✓ WASHING INSTRUCTIONS

The Comfort cover, the terry covers and the Flex arm support can be washed at up to 60°. N.B.: The Flex arm supports must be completely emptied before washing. Tip: Use liquid laundry detergent.

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