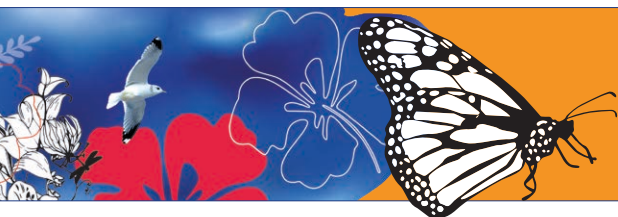
VIVO

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Preface

Dear user,

By choosing a Vivo mobility scooter you have chosen unequalled safety, quality and driving pleasure! We hope you will drive many comfortable and safe miles and trust that you will enjoy the freedom offered by the Vivo.

In this user manual you will find everything you need to know about the Vivo, from safety instructions and operating functions to maintenance tips. Carefully read the user manual, so you can get the most out of your mobility scooter.

We would like to keep you informed about updates, user experiences and other useful information. You can regularly read about the latest developments on the **www.life-mobility.com** website.

Kind regards,
Management and staff of Life & Mobility

Contents

Safety instructions	5
Introduction	12
Technical specifications	13
Product description	14
Settings and adjustments	17
Use	20
Maintenance	27
Transport and storage	30
Warranty	33
Disposal of product and packaging	34

Safety instructions

As the user of the mobility scooter, you are always responsible for complying with the locally applicable safety instructions and guidelines. It is important to first acquire driving experience under the supervision of an experienced instructor. Your dealer can advise you on this.

Life & Mobility accepts no liability for any loss or injury caused by negligence or by failure to comply with the safety instructions.

Always notify Life & Mobility and the relevant authority in case of a serious incident.

Attention:

- If you see this symbol in the manual, it indicates a supplementary instruction. Always follow this instruction. Negligence may adversely affect the operation of the mobility scooter and may cause physical injury or damage to the mobility scooter, the immediate vicinity or the environment.

General

Warning:

- *Never change the construction of the mobility scooter.*
- *Only use the mobility scooter for its intended purpose.*
- *Keep the mobility scooter away from open fire.*
- *Never allow children to play unsupervised near the mobility scooter.*

Safety instructions

Be careful:

- Use the basket only to transport objects.
- Do not use your mobility scooter to pull vehicles, carts, etc.
- Remember that there are protruding parts, both fixed and loose, in and on the mobility scooter. These can cause damage to the surroundings and the mobility scooter.

Attention:

- This mobility scooter is suitable for transporting a single person. Do not carry passengers.
- Only use original Life & Mobility parts and accessories.
- Never cover or remove the safety instructions and stickers.
- Always switch off the mobility scooter and take out the key before leaving it. This prevents unwanted use and/or theft.
- Use in rain, mist, black ice, snow and on salted roads may have an adverse effect on the electrical system.
- This mobility scooter does not come with lights as standard; therefore, do not ride it in the dark. Lighting can be supplied as an accessory.
- This mobility scooter does not have indicators, use hand-signals when changing direction.

Personal requirements

Warning:

- If you are under the influence of substances that may affect your ability to drive, you may not drive the mobility scooter.
- There are certain (medical) situations where it is desirable to first undergo driver-competence training supervised by a suitably equipped dealer.

Safety instructions

Attention:

- Please contact your doctor if you are taking prescription medicines or if you have physical limitations. Some medicines or limitations may affect your driving ability.
- Your vision must be adequate to drive the mobility scooter safely in any situation that may occur during use.
- Please follow the relevant traffic rules that apply.
- Make sure that you can always be seen by other road users. As a user of a mobility scooter you are very vulnerable on the road.
- Avoid distant routes. Make sure that help can be arranged in the event of an emergency.
- Switch off the mobility scooter if you are using your mobile telephone.

Assembling and adjustment

(see also the chapter 'Settings and adjustments')

Warning:

- Before using the mobility scooter, check that all components of the mobility scooter are properly mounted and secured.

Attention:

- Never remove the anti-tipping wheels.

Sitting in the mobility scooter

Warning:

- Make sure that your fingers and limbs do not get trapped. Always be very careful:
 - When folding away armrests.
 - If you are not in full control of your arms or legs.

Safety instructions

Driving the mobility scooter

(see also the chapter 'Use')

Warning:

- Please follow the relevant traffic rules that apply.
- Make sure that no loose clothing can get caught. These may become trapped in the wheels.
- Do not transport anything on the handlebars, instead, use the basket.
- Do not let anyone stand on your mobility scooter.
- Do not transport anything on your lap.

Attention:

- Before driving-off check that the mobility scooter is in the driving status and not in freewheel.
- Do not keep the magnetic key on your dashboard, but hang it from the steering column in the place intended for this purpose.

Slopes and other obstacles

Warning:

- Avoid slopes with a gradient greater than 6°. Driving on slopes (forwards or in reverse) is always dangerous.
- Be careful you do not overturn when driving on a slope. Never freewheel down a slope.
- Always drive straight up/down a slope, never angle across it.
- Drive carefully along (pavement) edges. An uneven surface increases the risk of overturning.
- Avoid prolonged driving on inclines. This can cause excessive wear on the engine and electrical components.

Maximum gradient

Approaching a curb

Safety instructions

Attention:

- Your centre of gravity moves when you drive up or down a slope. Please beware.
- Turn around carefully on a steep slope as there is a greater risk of overturning.
- Always approach a curb from directly in front.
- Never reverse over a curb or obstacle.
- The maximum curb height you can drive over is 5 cm.
- Soft, slippery or uneven surfaces increase the risk of a dangerous situation.
- The braking distance can be significantly longer on a slope than on a flat surface.

Transport the mobility scooter without passenger (see the 'Transport and storage' chapter)

Extremely dangerous:

- When transporting the mobility scooter in a motor vehicle, always make sure you are seated in a fixed seat of the motor vehicle. Never use your mobility scooter as a seat in a motor vehicle.
- You should not move the mobility scooter by using a hoisting device or a lifting system.

Warning:

- Make sure that your fingers do not get trapped when folding down the mobility scooter.
- Never carry the mobility scooter by the plastic parts of the housing.

Attention:

- Make sure that the mobility scooter is fully assembled before use.

Safety instructions

Miscellaneous

Warning:

- Do not put the mobility scooter in direct sunlight: the upholstery can become too hot, and cause burns or irritation to the skin.

Be careful:

- Never attempt to resolve a fault yourself. Contact your dealer instead.
- Do not allow the mobility scooter to come into contact with seawater: seawater is corrosive and will damage the mobility scooter.
- Do not allow the mobility scooter to come into contact with sand: sand may penetrate to the moving parts of the mobility scooter, causing faster wear and tear.

Attention:

A number of technical precautions have been implemented to further guarantee your safety:

- The immobiliser is activated whenever a battery charger is connected.
- In the event of an electronic fault, the mobility scooter will make a controlled stop.
- To avoid the risk of body parts becoming trapped, the necessary components have been guarded where possible.
- To secure your body in the correct position, your supplier can fit a hip belt to the fastening points of the hinge of the seat.

Safety instructions

EMC-approved

The mobility scooter is approved for EMC (Electro-Magnetic Compatibility). Tests have shown that mobility scooters are not sensitive or damaging to mobile communication equipment and electronically controlled equipment. Despite this, please be careful when using the mobility scooter and mobile or electronic equipment at the same time.

It is also not advised to use the mobility scooter close to strong electric or magnetic fields as can be found in industrial environments. If you notice problems, please switch off the mobility scooter.

Introduction

The Vivo is an electrically powered mobility scooter, suitable for both young and older people who have difficulty walking long distances.

This manual only describes the default settings of your mobility scooter. You will find the most up-to-date safety instructions and more information on our website: www.life-mobility.com

Recalls

If a product is being recalled, we will put a notice on our **www.life-mobility.com** website or you will receive a personal message from your dealer.

** Depending on the set speed, outside temperature, weight of the user, etc.*

Technical specifications

- Intended use: outdoors, between -25° C and + 50° C, on paved paths and roads
- L x w x h (seat in the highest position): 108 x 58 x 93 cm
- Weight (incl. batteries): 63 kg
- Weight of battery box: 15 kg
- Seat depth: 45 cm
- Seat width: 46 cm
- Seat height above footrest (adjustable): 42,5/45/47,5 cm
- Height between footrest and ground: 9 cm
- Max. driving distance with fully-charged batteries: 28 km* (18 miles approximately*)
- Max. (programmable) speed: 8 km/h* (5 mph*)
- Turning circle (radius): 118,5 cm
- Max. user weight: 120 kg (19 st)
- Battery capacity: 24 V (2x12V) / 22 Ah
- Max. load: 3A / 120/240 V / 50/60Hz
- Wheel size: 24x7.5 cm (9.4" x 3)
- Complies with: EN 12184 Class A
- Complies with requirements for resistance to ignition: ISO 7176-16

Service and technical support

Contact your dealer if you have any questions about specific settings, maintenance or repairs. Always have the serial number ready. You can find it on the CE sticker on the seat column on the mobility scooter.

Expected service life

Life & Mobility assigns a service life of 6 years to this product. This means that during this period, the product is fixable, safe and usable.

The statement above is only applicable if the user uses the product in the manner that Life & Mobility intended, marked with CE and including any original accessories.

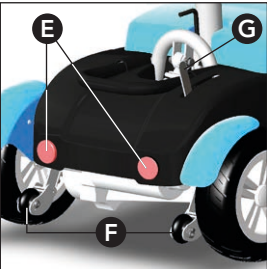
Product description

Checking the delivery

Before using the mobility scooter, check if the delivery matches the product description below. If you find any transport damage upon delivery, please inform your dealer. Life & Mobility trusts that the mobility scooter meets your expectations and hopes it brings you much convenience.

Product description


Mobility scooter product description


- | | |
|------------------------|----------------------------------|
| A. Folding backrest | I. CE sticker |
| B. Folding armrests | J. Battery box |
| C. Seat lever | K. Footrest |
| D. Handle | L. Front wheel |
| E. Reflector | M. Handlebars' rotary button |
| F. Anti-tipping wheels | N. Basket |
| G. Freewheel lever | O. Handlebars with control panel |
| H. Sprung rear wheels | P. Magnetic key holder |

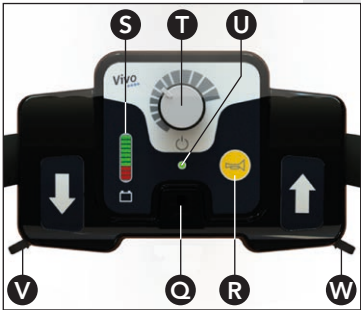
Product description

Product description control panel

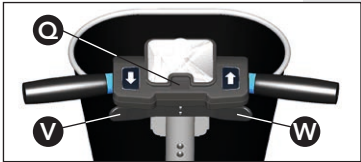Magnetic lock and thumb control lever

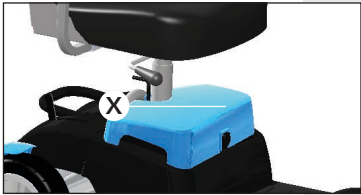Battery charger contact

Control panel

- Q. Magnetic lock / Magnetic key
- R. Horn
- S. Battery indicator
- T. Speed button
- U. Indicator on/off
- V. Reversing thumb control lever
- W. Forward thumb control lever
- X. Battery charger contact

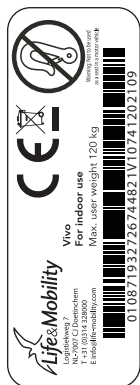
Not pictured

Various accessories are optionally available, including:

- Crutch holder
- Rear-view mirror
- Backrest bag
- Pneumatic tyres
- Seatbelt
- Lights
- Colour accents

See www.life-mobility.com for more accessories.

Product description

CE-sticker

Freewheel lever

Electric driving position

Push position

Markings on the mobility scooter

There are a number of stickers on the mobility scooter. Below is an overview of these stickers, with their location.

CE-sticker

The Vivo complies with the mandatorily applicable European REGULATION (EU) 2017/745 for Medical Devices and has a CE label.

The CE sticker can be found on the seat column on your mobility scooter.

Freewheel sticker

The sticker above the freewheel lever (G) indicates the "push position". In this setting the mobility scooter can be moved freely.

The sticker below the freewheel lever indicates that the mobility scooter is ready to be electrically driven.

A CE label is attached to the steering column, indicating the maximum user weight and that it is a medical device [MD].

You are also reminded to read the manual.

Settings and adjustments

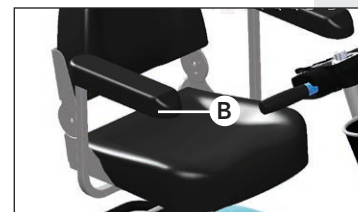
The user can define settings and make adjustments. If you have any doubts about your skills, please contact your dealer.

Adjust seat width

1. Loosen the socket screws (a) on the bottom-rear of the seat.
2. Set the width to suit your personal requirements.
3. Re-tighten the socket screws.
The arm rests are now locked in the current position.

Changing the seat width

Armrests

Armrests

Fold the armrests (B) up to make getting-out easier.

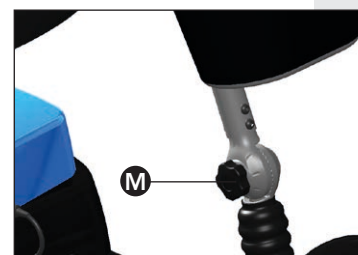
Adjusting the steering angle

Attention:

- Do not adjust the angle of the steering column while driving.

Adjust the steering column so that you can easily reach the controls and the display. The steering column can be adjusted by a toothed-wheel.

Change steering angle

1. Loosen the rotary button (M) at the bottom of the steering column.
2. Adjust the steering column to the required angle.
3. Again tighten the rotary button; ensure that the teeth match each other.
The steering column is now locked in place.

Settings and adjustments

Adjusting the seat height (3 positions)

1. Remove the seat (see page 29).
2. Remove the battery box; use the handles on both sides for this.
3. Loosen the bolt and adjust the seat column to the required seating height.
4. Replace the battery box.
5. Replace the seat.

Dismantling

The mobility scooter can easily be dismantled for transport or storage (see the 'Transport and Storage' chapter also).

Warning:

-  Be very careful when dismantling the mobility scooter. Ensure that your hands and fingers do not get trapped in the hinge system.

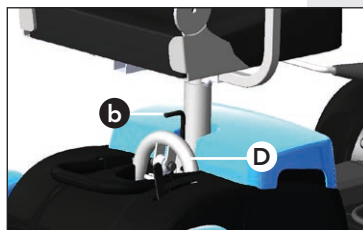
Please note:

-  The mobility scooter must be secured for transport after disassembly at the indicated places.
- The seat can be grabbed from the seat itself and the back.
 - The front part of the mobility scooter has grab places on the bumper and the seat column.
 - The rear part of the mobility scooter has a lifting bracket.
 - During transport, keep the battery separate and do not replace it.



1. Remove the seat.
2. Remove the battery box; use the handles on both sides for this.
3. Pull the unlocking lever (b) on the rear side of the mobility scooter vertically upwards.
4. At the same time, press the handle (D) towards the ground.


Disassembled mobility scooter

Unlocking

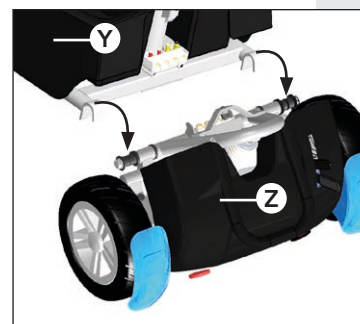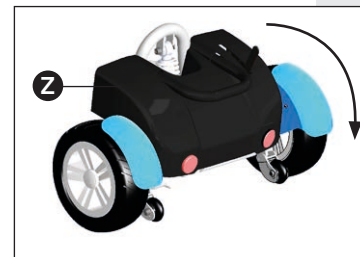Determining the position

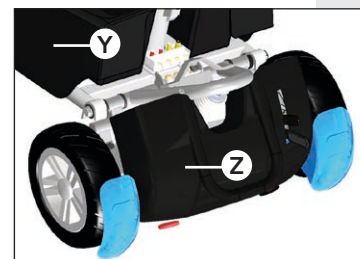Assembling

5. Now, lift the front section (Y) away from the rear section (Z).

Assembling

1. Tilt the rear section (Z) of the mobility scooter at an angle of 45 degrees onto the anti-tipping wheels.

2. Attention:

- Determine the hook-in position accurately to prevent damage to the body shell.
3. Lift the front section of the mobility scooter and carefully hook it onto the frame tube of the lower rear section.

4. Allow the front section to lower gently, until both sections audibly click together.
5. Place the battery box in the space provided

Advance checks

Check the following before driving off:

- The battery status on the display.
- The brakes.
- Whether the freewheel lever is pointing down (electric driving position).
- Whether the battery charger has been removed.
- Whether anything is under the mobility scooter.
- If you notice any unusual behaviour in your mobility aid (noise, odour, behaviour), please contact your dealer immediately.

Getting in and out by yourself

1. Push the seat lever (C) up.
2. Keep the seat lever pushed up and turn the seat a quarter turn to the left or right.
3. Get in or out.
4. Release the seat lever.
The seat is now locked in place.

Getting in and out


**Attention:**

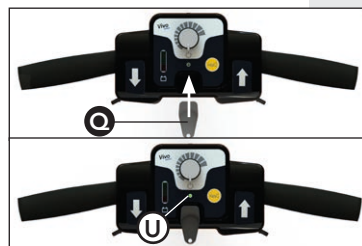- Make sure that the seat is properly locked in place before driving off.

Switching on

1. Use the magnetic key (Q) to switch on the Vivo.

*The on/off indicator light (U) will light up.
The mobility scooter is now ready to drive.*

Green on/off indicator light

The LED bar lights up

2. Hang the magnetic key from the steering column in the place intended for this purpose.

3. Turn the speed button (T) to select the required speed.
The battery's LED bar will light up.

**Attention:**

- Choose a suitable speed setting for each situation.
- Turn the speed button (T) to the left to drive slower.
- Turn the speed button (T) to the right to drive faster.

Maximum speed

The maximum speed ex-factory is 7,5 km/h but your dealer can program a lower speed.

Driving forward and reversing

1. Push in the right thumb control lever (W) slowly (in the direction of the handlebars).
The mobility scooter starts moving forwards.
2. Push in the right thumb control lever further.
The mobility scooter will start moving forwards more quickly.
3. Release the right thumb control lever.
The mobility scooter will stop.
4. Use the left thumb control lever (V) to reverse.

Thumb control levers

**Attention:**

- If the mobility scooter is not used for 5 minutes, the mobility scooter will automatically switch to the standby mode for 50 minutes. The battery's LED bar flashes to indicate that the mobility scooter is still switched on. After 50 minutes of being in the standby mode, the mobility scooter will automatically switch off.

The battery's LED bar will flash

Driving again after being in the standby mode

1. Turn the speed button back to the start position.
The LED bar will turn off

Speed button in the start position

Switch off with
magnetic Lock

2. Next, turn the speed button to select the required speed.
3. Use the thumb control lever to drive again.

Switching off

1. Turn the speed button (T) to the left to the start position.
2. Use the magnetic key (Q) to switch off the Vivo.

The on/off indicator light and the battery's LED bar will turn off. The mobility scooter will now have been switched off.

Attention:

After 50 minutes of being in the standby mode, the mobility scooter will automatically switch off. You now need the magnetic key to again switch on the Vivo.

Attention:

- Adjust your speed using the thumb control lever. The speed setting determines how fast you can go if you push the thumb control lever all the way.
- Check that the road behind you is clear when reversing.
- The mobility scooter reverses more slowly than when it drives forward.

Warning:

- Adjust your speed before steering into a bend. The mobility scooter may topple if you drive into a bend at top speed.
- Despite the small turning circle of the mobility scooter, it may still be difficult to turn into lifts or doorways. Make sure that you always use the correct speed setting for manoeuvring.

Batteries

Warning:

- Always have your dealer perform maintenance to the batteries. Never open the battery box yourself.
- Never connect a cable to the AC/DC converter or the battery charger. Contact with components may cause electric shock.
- Only use the charging plug and the supplied battery charger of Life & Mobility.

Be careful:

- New batteries must be run in. The full capacity of batteries is only reached after they have been used and charged a couple of times. Take this into account during your first few trips.
- By preference, charge the batteries on a daily basis; this will extend the service life. In any case, at least charge the batteries once a month.
- Always charge the batteries to full capacity.
- Avoid draining the batteries completely. The mobility scooter will lose power which will have a negative effect on the service life of the battery.
- Charge fully drained batteries as soon as possible using a 24-hour cycle.

Attention:

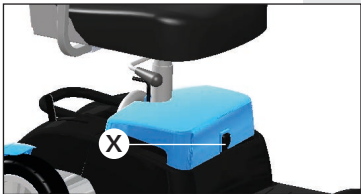
- At lower temperatures the capacity of the batteries is reduced. This also reduces the operating range.
- Always connect the charger when putting the mobility scooter away for the night.
- You cannot overcharge the batteries supplied. Thanks to a voltage limiter, the charger automatically switches off when the batteries are full.

Use

Battery indicator

Battery charger contact

Reading the battery status
The battery indicator (S) on the control panels shows the remaining battery capacity:

Battery status	Indicator
All red and green blocks are visible.	The batteries are full.
3 red blocks are visible	You can drive another 2 - 4 miles (3 - 6 km) (depending on the type of battery and other circumstances). You should start charging now.
1 block visible	Recharge the mobility scooter as soon as possible.

- Charging the batteries*
1. Insert the battery charger plug into the battery charger contact (X).
 2. Insert the plug of the mains cord in a wall socket.
 3. Always fully complete the charging cycle of 8 - 10 hours. The batteries are fully charged when the green lamp is lit

Error codes
When there is a error message, the LED Bar will flash.

Error message	Indicator
1 block flashes	The battery is empty or connected incorrectly. Check whether the battery is connected correctly. Then try to charge the battery again.

Use

Error message	Indicator
2 blocks flash	The electrical connection to the engine has been interrupted. Have your supplier check all engine electrical contacts.
3 blocks flash	Short circuit between engine and battery. Contact your supplier.
4 blocks flash	The freewheel is engaged or the brake has been disengaged manually. Check the position of the freewheel lever.
5 blocks flash	Not in use.
6 blocks flash	The brake has intervened. For example because the charger is still connected or the seat is not in the correct driving position.
7 blocks flash	Error with regard to the throttle. Switch off the mobility scooter and switch it back on. Contact your supplier if the problem persists.
8 blocks flash	Electronics error. Contact your supplier.
9 blocks flash	The freewheel is engaged or the brake has been disengaged manually. Check the position of the freewheel lever.

Error message	Indicator
10 blocks flash	Too high voltage. Contact your supplier.
Blocks run up and down from 1 to 10 and back.	Throttle operated when switching on. Release the throttle, switch off the mobility scooter and switch it back on. Contact your supplier if the problem persists.

Freewheel lever

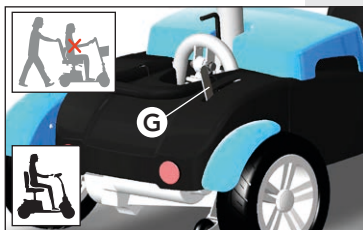
Warning:

- Never use this function on a slope or while driving. It will disable the brakes.
- Always push the freewheel lever downwards to re-engage the brakes after using the freewheel function.

1. Push the freewheel lever (G) up to move the mobility scooter by hand.
2. Push the freewheel lever down for the electric drive position. Teach yourself to do this each time you have used it.

Parking

When parking your scooter, place a chain lock through one of the wheels for added security.

Freewheel lever

Cleaning

Be careful:

- Do not use bleach or solvents.
- Do not use a high-pressure cleaner to clean the mobility scooter.

Attention:

- Regularly clean the upholstery and the frame.

Upholstery

1. Remove any light dirt with a solution of a mild cleaning agent and warm water.
2. Carefully scrub the upholstery with a soft brush if the dirt has been absorbed into the upholstery.

Frame

1. Clean the housing with a non-abrasive cleaning product.
2. Apply non-abrasive car wax to the entire frame.

Batteries (see also the chapter 'Use')

Attention:

- Always fully charge batteries before using them for the first time.
- Be careful with damaged batteries. Leave (environmentally-friendly) replacement of the batteries to your dealer.

Be careful:

- Batteries can be dangerous. Only allow your batteries to be installed and maintained by a qualified scooter mechanic. They have the required qualifications and tools to perform the installation and maintenance safely and correctly.

Maintenance

- The mobility scooter operates at 24 V and uses 2 12 V batteries connected in series. Never use both batteries at the same time:
 - Batteries from different manufacturers
 - Batteries with different technologies
 - Batteries with different date codes

Tyres (if pneumatic tyres)

Attention

- Contact the supplier in the event of a flat tyre.
- The tyre pressure may not be more than 3.45 bar.
- The pressure of the front tyres must be at least 2.0 bar.
- The pressure of the rear tyres must be at least 2.5 bar.

Service and technical support

At least once a year have the mobility scooter checked and serviced by your dealer. Preventive maintenance ensures that the proper and correct functioning of all moving parts is checked, as well as the electronics and the condition of the batteries.

Repairs should only be performed by your dealer or by Life & Mobility, and only original parts supplied by Life & Mobility should be used for this.

Maintenance matrix

The following matrix indicates which parts should be regularly checked. If a defect occurs, contact your dealer to resolve this issue.

Maintenance

Part	Action	Daily	Weekly	Monthly	Each quarter	Annually
Battery charger	Check functioning	X				
Housing	Clean with damp cloth		X			
Tyres	Check tyre pressure			X		
	Check profile depth (min. 1.6 mm)				X	
Batteries	24-hour charging cycle			X		
Mobility scooter	Servicing					X
Frame	Check for rust, deformations and other damage, carry out repairs if necessary.					X

Servicing by the dealer

The following items must be checked when being serviced:

1. Check parts to determine whether they are broken. If broken, the part must be replaced.
2. Check parts on rust. If required, touch up with metal paint or replace .
3. Check moving parts on clearance. If the clearance has a negative effect on operation, adjust or replace the relevant part.
4. Check the bolt connections on tightening torque and apply, if required, the overview below;
M5 screws -> 6 Nm
M6 screws -> 10 Nm
M8, M10 and M12 screws -> 25 Nm

Transport and storage

Transporting the mobility scooter

Extremely dangerous:

- When transporting the mobility scooter in a motor vehicle, always make sure you are seated in a fixed seat of the motor vehicle. Mobility scooters may never be used as seating support in a motor vehicle, even if the mobility scooter is secured with a taxi restraint system.
- You should not move the mobility scooter by using a hoisting device or a lifting system.

Be careful:

- Never carry the mobility scooter by the plastic parts of the housing.

Attention:

- The parts are heavy. Ask for help if required:
 - Weight of the battery box: 15.5 kg.
 - Front part of the mobility scooter: 18.5 kg.
 - Rear part of the mobility scooter: 16.1 kg.
 - Seat: 11 kg.
 - Basket: 0.6 kg.

Removing the seat

1. Stand next to the mobility scooter.
2. Push the backrest forward.
3. Turn the lever (C) downwards.
4. Rotate the seat a quarter turn to the left or to the right.
5. Lift the seat from the mobility scooter.

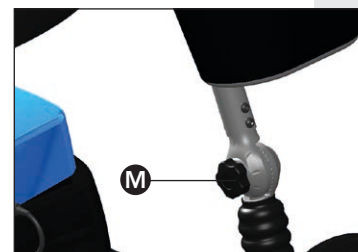
Removing the seat

Transport and storage

Folding down the steering column

1. Loosen the button (M) at the bottom of the steering column.
2. Move the steering column towards the seat as far as possible.
Tip: if the seat is removed, the steering column can be folded down further.
3. Re-tighten the lever (C).

Folding down the steering angle

Transport

Once the seat has been removed you can transport the mobility scooter in the boot, for example.

It may be the case that your mobility scooter is too large for some public means of transport. Make your mobility scooter smaller if possible.

If required, the mobility scooter can be further dismantled, see chapter 'Settings and adjustments'.

Storage when not using the mobility scooter

Attention:

- Always store batteries fully charged.
- Fully charged AGM batteries will retain their charge for up to six months, but they must always be charged before use. The battery capacity will only reach its full capacity after a few times of using and charging.
- Avoid storage of batteries at extremely high or low temperatures.
- AGM batteries do not need to be topped up with water. Contact a qualified scooter mechanic if you have any performance issues.

Warranty

Longer than a week:

- Connect the battery charger at least once a week to maintain the condition of the batteries or put the mobility scooter away for the night with the battery charger connected.

Very long term (e.g. during winter or due to illness):

- Have your dealer disconnect the wiring of the batteries.

Terms and Conditions of Warranty

The Life & Mobility products are made with care and are carefully inspected before leaving the factory. Should it emerge that a product does not meet expectations please contact the party where you purchased the product.

Life & Mobility respects the laws and regulations that protect the consumer and that apply in the country in which the product is purchased.

The warranty does not apply to defects caused by: failure to observe the instructions for use and maintenance, use other than normal use, wear and tear, negligence, overloading, third-party accidents, non-original parts used and defects not caused by the product.

This warranty replaces any other warranty, either stipulated by law or communicated verbally, except for anything guaranteed in writing by Life & Mobility.

Warranties are only valid within the EU.

Disposal of product and packaging

Disposal of product and packaging

All parts of your mobility scooter are recyclable. Your dealer will know the collection points for disposal of the mobility scooter.

From an environmental perspective, we recommend that you dispose of the packaging and the mobility scooter by separating the waste as much as possible, allowing the waste materials to be recycled efficiently.

This manual has been prepared with the utmost care and is based on information known to Life & Mobility at the time of publication.

The information in this manual applies to the standard version of the product. Life & Mobility does not accept responsibility for any damage that may occur due to changes or modifications to the product after selling it for the first time. Neither can Life & Mobility be held liable for any damage caused by third-party work.

The information in this manual may not be reproduced and/or made public by any means without prior written permission from Life & Mobility.

Pursuant to the Trade Names Act, the trade name and trademarks used by Life & Mobility may not be considered free.

No rights can be derived from this text. Subject to changes.

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© Life & Mobility – All rights reserved